



# **Technical Data Sheet**

# Filament-Oss HA

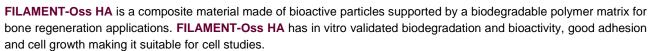

**FILAMENT-Oss HA** is bioabsorbable, can be sterilized by ultraviolet light, has antibacterial effect and allows obtaining multidirectional and customized porosity.

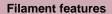

Particle	Hydroxyapatite	
Polymeric matrix	PLA	
Particle loading (wt.%/vol.%)	40 wt.%/ 20 vol.%	
Diameter	1.75 ± 0.15 mm	
Density	1.63 g/cm <sup>3</sup>	
Linear Density	0.039 g/cm	
Format	Spool vacuum packed	

#### **Thermal Properties**

Glass Transition Temp.	62 °C
Melting Temp.	156 °C
Degradation Temp.	320 °C

### **Printing Recommendations**

Printing Temp.	155-165 °C
Hot Pad	40 °C
Printing Speed	15-30 mm/s
Layer Height	> 0.2 mm
Nozzle Diameter	> 0.4 mm
Head travel speed	< 150 mm/s
Stand-by Temp.	30 °C


#### **Storage Conditions**

Keep in dry place	
Protect from direct sunlight	
Storage between 5°C- 30°C	


## **Specific properties**

Osteoinductive and osteoconductive properties for	
bone regeneration.	
Sterilized by UV.	
Controlled PLA degradation rate.	

#### Thermal behavior

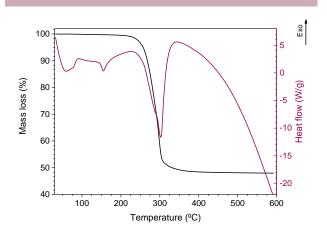

#### Filament cross-section

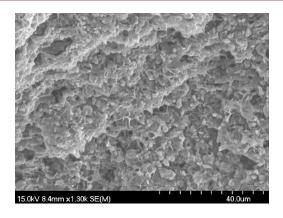



Samples printed with Filament-Oss 20HA



# **Powder Specifications**

# Hydroxyapatite Powder


#### **Identification Product**

Commercial name	CAPTAL® Sintered Hydroxyapatite (1300 °C)	
Chemical formula	Ca <sub>5</sub> (PO <sub>4</sub> ) <sub>3</sub> OH	
Supplier	Plasma Biotal Limited	
Characteristics/ Description	Medical-grade, synthetic, high- purity, highly crystalline bioresorbable bone substitute material designed to closely mimic the properties and composition of natural bone.	

## **Chemical composition**


Ca/P ratio	1.74	X-Ray
Purity	(99.5-99.8) %	Fluorescence
Density	3.16 g/cm <sup>3</sup>	Helium picnometry
Spec. Surf. area	1.4 m <sup>2</sup> /g	N <sub>2</sub> adsorption- desorption

#### Particle morphology

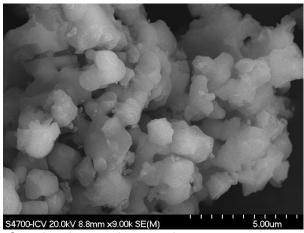

Scanning electron microscope image

## Particle size distribution

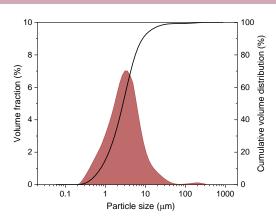

 $D_{10}$ : 0.75 $\mu$ m  $D_{50}$ : 2.76  $\mu$ m  $D_{90}$ : 8.6  $\mu$ m Measured by Laser Diffraction at small angles

Disclaimer: The technical information contained in this data sheet has been obtained by laboratory characterization and should be treated as a reference. Any description, data or printing recommendations given in this document may change without prior information and do not constitute the agreed contractual quality of the product. Printing conditions may vary depending on the 3D printer model used and part printed. Due to the numerous factors that may affect the data shown, we make no warranty of any kind, express or implied, as to the properties of the product or its suitability for a particular use.